

# + How to use PROSHIELD<sup>◇</sup> PLUS Skin Protectant

**Smith+Nephew**

PROSHIELD<sup>◇</sup> PLUS  
Skin protectant

## Application: on injured skin



- 1 Thoroughly cleanse the entire skin area with a suitable cleanser (e.g. PROSHIELD Foam & Spray Incontinence Cleanser)



- 2 Wash or cleanse hands, remove ointment from tube



- 3 Apply a **liberal, thick** coating

## Application: on intact skin



- 1 Thoroughly cleanse the entire skin area with a suitable cleanser (e.g. PROSHIELD Foam & Spray Incontinence Cleanser)



- 2 Wash or cleanse hands, remove ointment from tube



- 3 Apply a **thin** layer

## Removal

PROSHIELD PLUS Skin Protectant should be re-applied after cleansing when the ointment or sheen is no longer visible, on the surface of the skin

**Step 1** To remove, simply spray the PROSHIELD Foam & Spray Skin Cleanser directly onto the affected area<sup>1</sup>

**Step 2** Gentle wipe clean and pat dry (no rinsing is required)<sup>2-6</sup>

For more information contact your local sales representative

Watch the PROSHIELD PLUS Skin Protectant application video on our **Smith+Nephew Wound YouTube channel** today!



For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

### PROSHIELD PLUS Skin Protectant ordering code

Size	Code	PIP Code	NHSSC Code
PROSHIELD PLUS Skin Protectant 115g	8213-0300-04	359-8760	ELZ537

Smith & Nephew  
Croxley Park, Building 5,  
Hatters Lane, Watford,  
Hertfordshire WD18 8YE

[www.smith-nephew.com/uk](http://www.smith-nephew.com/uk)  
T +44 (0) 1923 477100  
F +44 (0) 1923 477101

◇ Trademark of Smith+Nephew,  
All Trademarks acknowledged  
© September 2021  
Smith+Nephew 31638-uk

**References** 1. Smith+Nephew 2019. Verification Report for Removal of PROSHIELD Plus Skin Protectant. U/064/R2. 2. Smith+Nephew 2019. Claims Support - "Gentle / Low Irritation" and "nonirritating and non-sensitising". Internal Report. PSS390. 3. Smith+Nephew 2020. Review of certificate of analysis for PROSHIELD FOAM & SPRAY Incontinence Cleanser. Internal Report. RD/20/015. 4. Wall L, Vernon T. Launch of a Skin Care Regime to Reduce Incidence of Moisture Associated Skin Damage. Paper presented at: Tissue Viability Conference; 2016; Cardiff, UK. 5. Ling L. Proshield skin care protective system: A sequence of evaluations Paper presented at: Wounds UK; 2011; Harrogate, UK. 6. Flynn D, Williams S. Barrier creams for skin breakdown. Nursing & Residential Care. 2011;13(11):553 - 558.